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
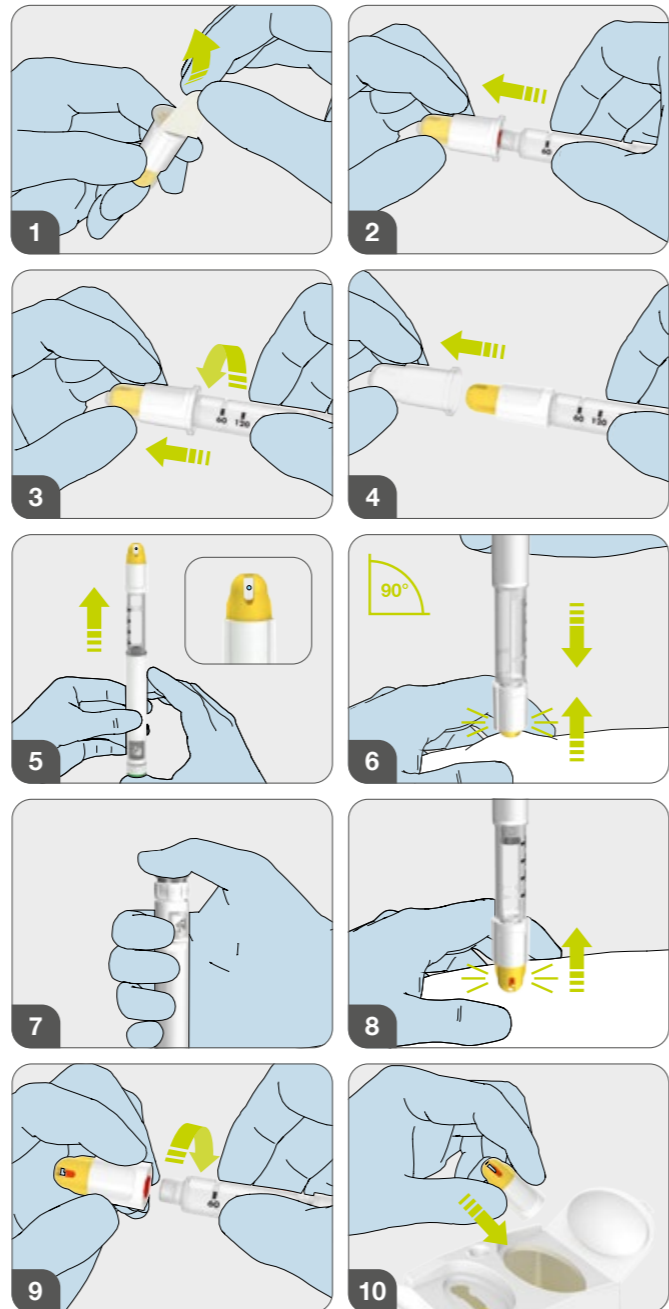
**Do not/Nicht/Ne pas/Non/No/Niet doen/ikke/ikke/Älä/Inte:**

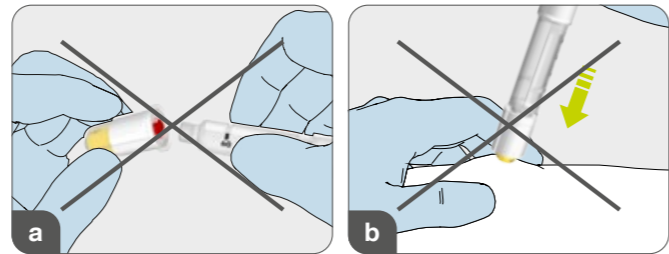**en**

**Indications for use/intended use:**  
 mylife™ AutoProtect™ PRO Dual Safety pen needles are disposable, sterile needles intended for use with a pen injector device for the subcutaneous injection. After injection and withdrawal from the body, the safety shield automatically covers the needle to minimize the risk of accidental needle sticks.

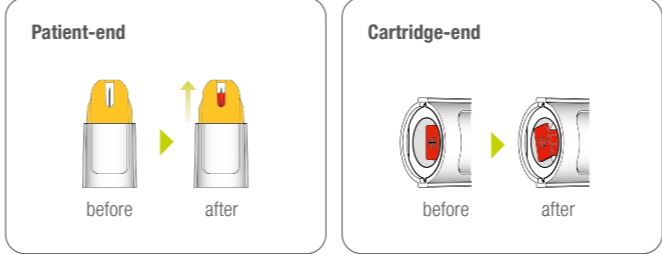
**Caution:**

- Safety pen needles are for self-injection and professional use.
- Safety pen needles are single use only. The locking mechanism prevents reuse and cross contamination.
- Do not use if the sealing film is missing or the needle is bent or damaged.
- Avoid contaminating the Safety pen needle with hand lotion, oils or dirt.
- Do not use expired pen needles.
- Speak to your doctor before changing the length of the needle or injection site.
- Professional health care personnel should follow the specific directions for health care personnel.
- The pen needle is sterile as long as the protective tab remains intact. After every injection, please unscrew the pen needle from the insulin pen and dispose it in a sharps container.
- Consult the local health regulations for appropriate disposal. Store in a cool, dry place away from chemical substances.
- Notification of any serious event related to the device shall be reported to the user of the device, and (or) manufacturer and competent authorities of member state where the patient is located.

**Instructions for use:**

- 1 Remove the sealing film from the Dual Safety pen needle.
- 2 Insert the pen into the Safety pen needle horizontally.
- 3 Attach the Safety pen needle securely to the pen injector device by twisting clockwise until it stops.
- 4 Remove the cover from the Safety pen needle to expose the needle.
- 5 Prime the pen injector device according to the pen injector device instructions for use and set the correct dose.
- 6 Inject into the skin as recommended by your physician with or without lifting a skinfold.
- 7 Hold the injection for at least 10 seconds before withdrawal. Count slowly to 10 seconds.
- 8 Remove the Dual Safety pen needle by lifting it straight up from the injection site.
- 9 Rotate the pen counterclockwise.
- 10 Discard it into a sharps container.

- a DO NOT hold at an angle when inserting into place.
- b DO NOT inject non-vertically.

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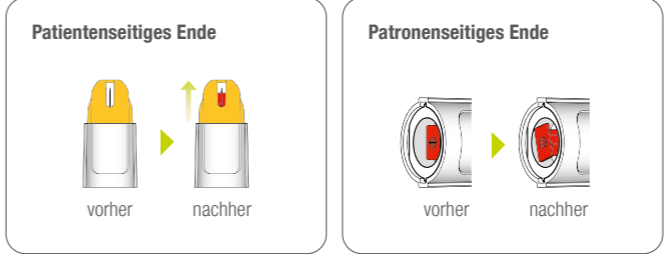
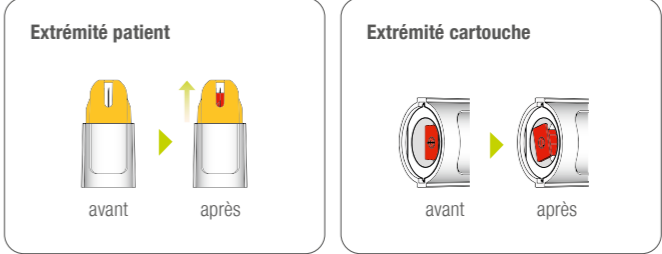
10294661/001 // 2022-09

0.25 mm x 5 mm - 31G  
 0.33 mm x 8 mm - 29G

**Genray Hangzhou Life Technology Co., Ltd.,**  
 Room 606, 6th Floor, Building 1, No. 1155 Dayuan Road, Qingshanhu Street, Lin'an District, 311308 Hangzhou, Zhejiang, China

Manufactured for  
 Ypsomed AG  
 Brunnmattstrasse 6  
 3401 Burgdorf, Switzerland  
 www.mylife-diabetescare.com

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